

## INSTRUCTION

### PRODUCT:

## The filtering half-mask EN 149 FFP2 NR

The Declaration of Conformity is available at: [www.r-zero.it](http://www.r-zero.it)

The filtering half-mask is a complete respiratory protection equipment and is intended to protect the user against the harmful effects of air pollution in the form of solid and/or liquid particles forming aerosols (dusts, fumes and mists).

The deadline for the use of the product is found on its packaging.

A damaged half-mask and the one whose storage data has been exceeded cannot be used.

Class: *FFP2 (according to UNI EN 149:2009)*

The indication *NR* in the marking indicates that the filtering half-mask is not reusable and must be thrown away at the end of the work shift. Comfortable and durable throughout the work shift (8 hours).

### Instruction

|   |  |
|---|--|
|  | Take the filtering half-mask out of the package and stretch it.  |
|  | The nose clip part is facing up, the side without nose clip is facing the face, and the ear straps are pulled on each hand.    |
|  | Face the mask against the chin first, and pull the ear band back to the ear and hang it on the ear until it feels comfortable. |
|  | Adjust the filtering half-mask to fit the face, and mold nose clip around your nose to achieve a secure seal.                  |



If it is impossible to obtain a protective association with the profile of the face, do not enter areas of potential risk.

Failure to comply with the prescriptions prescribed in this note may lead to incorrect use of the PPE and cause illness and death.

A properly chosen respirator is essential to protect your health. Before using the PPE, consult a person trained in industrial hygiene or personnel experienced in occupational safety to determine whether or not it is suitable for use.

This product does not supply oxygen. Use only in protected ventilated areas with sufficient oxygen content to sustain life. The limits concerning the presence of oxygen in a given context for the impression of these masks are regulated by country, a distant reference is invited to the parameters defined in the country of use. R.Zero srl however, it is not recommended for use in environments with oxygen concentrations lower than 19.5%. Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use in an atmosphere where there is a danger of explosion. Leave the work area immediately if breathing becomes difficult, or there is a sense of anguish or dizziness. Beard hair and respiratory ailments can reduce the efficiency of these filtering half-mask. Do not modify or alter the filtering half-mask in any way. Single use only. It requires no maintenance. Discard the half-mask after single use.

Store unused half-masks in their unopened packaging in an environment away from sunlight.